Spiked-in tests of cancer cell lines for the platform development of circulating tumor cells enumeration

Sung Man Yoon¹, Jae Hyuk Lee¹, Young Woong Sohn¹, Yoon Kim¹, So Jung Choi¹, Dong Hyoung Lee¹, Jong Kil Lee², Sung Han Rhim³, Byung Hee Jeon^{1,*}

¹ Cytogen Inc., Seoul, Korea,

² Department of Mechanical Engineering, Korea Polytechnic University, Siheung, Kyonggi-do, Korea ³ Department of Mechanical Engineering, Dankook University, Yong-in, Kyonggi-do, Korea







Circulating tumor cells (CTCs)

- \succ Circulating tumor cells (CTCs) present in the peripheral blood of cancer patients, and the numbers of CTC are very low.
- \succ It has been known that CTCs initiate the metastasis of tumor, thus the significance of CTCs is getting attentions.
- \succ The numbers of CTCs are associated with prognosis and clinical outcomes in various types of cancers.

Cytogen's CTC enumeration platform

> The CTC enumeration platform is an automatic counting system based on image processing algorithm using cell size, cell surface area, area intensity, maximum intensity, circularity.

The purpose of this study

 \succ In this study, we developed the CTC enumeration platform and tested the performance of the platform using spiked-in tests with PC9 cells.

Cell DATA Analysis & Enumeration results

- \succ From 10 individual experiments, the average number of cell counting was 98 $(\pm 0.49).$
- \succ The cell Intensity, cell diameter, cell area showed characteristic of cell morphology



MARERIALS AND METHODS

Blood Sample Collection

- \succ PC9 cells were spiked-in 7.5 ml of blood from healthy volunteers, and the blood samples were processed through the size-based filtration.
- The recovered cells were stained for EpCAM and CD45



Figure 1. Production process of slide glass(Type : FRONTIER)

Imaging Detection & Enumeration Algorithm





Figure 5. Comparison of two cell enumeration methods (automatic and manual counting)

> The images were photographed by 3-channel fluorescent filters(UV2A, FITC,

G2A) of the optical-based image analyzing system(Cytogen's CTC enumeration

platform).



Figure 2. Imaging detection process

Figure 3. CTC enumeration algorithm

② VERIFICATION

Target

20 25 30 35 40 45 50 55 60 65 70 75 80 85 90 95 10



- We developed the CTC enumeration platform and tested the performance of the
- platform using spiked-in tests with PC9 cells.
- \succ These results showed the reliability of the CTC enumeration platform.
- \succ The CTC enumeration platform may provide the potential applicability in CTC

counting of patients' samples.